

PACS Keyboard Shortcuts

Annotations		Layout		Refresh	Ctrl+R
Arrow	A	Cycle Series Backward	Ctrl+Left Arrow		
Cobb Angle	Num/	Cycle Series Forward	Ctrl+Right Arrow	WorkFlow	
Ellipse	C	Drag and Drop Image Viewports	Mouse Left Drag	Close Study	NumMinus
Elliptical ROI	R	Layout 2x2	Shift+V	Close Study as Dictated	NumAdd
Eraser	E			Search by Patient Name	F4
Intensity Analysis	=	Link		Show Unreported Studies	F3
Label Disk	D	Link All	L	Window Level	
Label Vertebrae	V	Precise Registration	P	Default	Num0
Linear Distance	M	Quick Registration	Q	Estimate	NumPeriod
Show/Hide Annotations	Ctrl+A	Remove All Registration	X	Abdomen	1
Simple Angle	N	Unlink All	U	Lung	2
Text	T			Liver	3
		Miscellaneous		Mediastinum	4
Flag	Space	Display DICOM Header	H	Ortho Bone	5
		Invert Images	I	CT Angio	6
Image Viewport		Toggle Text	Ctrl+T	Soft Tissue Abdomen	7
Display Zoom Window	Z	Toggle bitmap overlays	Ctrl+Alt+B	Neuro Spine Bone	Num1
Jump to End	End	View Study Details	Ctrl+D	Head SDH	Num2
Jump to Middle	Insert			Soft Tissue Neck	Num3
Jump to Start	Home	Orientation		Head	Num4
Page Backward	Page Up	Flip Horizontally	/	Neuro Ortho Bone	Num5
Page Forward	Page Down	Flip Vertically	\	T1 Brain	
Row Backward	Left Arrow	Rotate Clockwise	. (period)	T2 Brian	
Row Forward	Right Arrow	Rotate Counter-Clockwise	, (comma)	Vertebrae	
Select Multiple Images	Mouse-Ctrl+Left Click			Zoom / Pan	
Select Single Image	Mouse - Left Click	Patient Documentation		Interactive Pan	Mouse Shift+Left Drag
Show Next Stage	Down Arrow	New Report	Ctrl+N	Interactive Zoom in Place	Mouse Ctrl+Wheel Scroll
Show Previous Stage	Up Arrow	View Report	Ctrl+V	Show Pan Tool	Shift+P
		Presentations and Bookmarks		Show Zoom Panel	Shift+Z
Index Viewport		Load Bookmark	Alt+B		
Play All/Stop All toggle	Ctrl+Shift+P	Load Presentation	Alt+P	Magnifying Glass	
Play/Stop Toggle	Ctrl+Shift+T	Save Bookmark	Ctrl+B	Shift+M	
Switch to Axial View	Alt+A	Save Presentation	Ctrl+P		
Switch to Coronal View	Alt+C			Post Processing	
Switch to Sagittal View	Alt+S			Shift+O	

Main Tool Bar

Study List
Folder Finder
View Report
Create Report
Overlays
Lines
Teaching
Send
Disk Import/ Export
Refresh
Sweep
Quit
Preferences
Help

Right Click Menu

Select
Annotate
Window/Level
Link
Pan
Zoom
Re-Orient
3D
3D/MPR
Mirada Fusion
Layout
Study Info
Cine
Reset