


		1.5T	15.0	SI Joints for SACROILIITIS MRI - UW MADISON			7/03/07a Tuite/Desmet/Robichaud			
<b>8ch CARDIAC coil</b>										
	<b>GRx for Cor views</b>			<b>GRx for Ax views</b>						
<b>1</b>		<b>2</b>	<b>3a</b>		<b>4</b>	<b>5</b>	<b>6</b>	<b>Give</b>	<b>+C 7</b>	<b>+C 8</b>
<b>3 Plane loc</b>		<Cor T1 FSE	<Cor FSTIR		Ax T1 FSE	Ax T1 FSE	Ax T2 dk fat FSE	<b>IV contrast</b>	<Cor T1 dk fat FSE	Ax T1 dk fat FSE
					<b>fatsat</b>	<b>fatsat</b>			<b>fatsat</b>	<b>fatsat</b>
2d std loc		2d FSE-XL	2d FSE-IR		2d FSE-XL	2d FSE-XL	2d FSE-XL		2d FSE-XL	2d FSE-XL
		SCIC on			SCIC on	SCIC on			SCIC on	SCIC on
		NP, TRf ZIP 512	SEQ, NP ZIP 512		NP, TRf ZIP 512	NP, TRf ZIP 512	NP, TRf ZIP 512		NP, TRf ZIP 512	NP, TRf ZIP 512
		Fast	Fast		Fast	Fast	Fast		Fast	Fast
		TE MF TR 600	TE 44 TR 3075		TE MF TR 600	TE MF TR 600	TE 68 TR 2200		TE MF TR 600	TE MF TR 600
		ETL 4 BW 20	T1 160 ETL 8 BW 12		ETL 4 BW 20	ETL 4 BW 20	ETL 6 BW 20		ETL 4 BW 20	ETL 4 BW 20
					<b>Sat scr FAT</b>	<b>Sat scr FAT</b>			<b>Sat scr FAT</b>	<b>Sat scr FAT</b>
	<b>Usr CV</b>	<b>Usr CV</b>		<b>Usr CV</b>	<b>Usr CV</b>	<b>Usr CV</b>		<b>Usr CV</b>	<b>Usr CV</b>	
	Blur can 1 x	Blur can 1 x		Blur can 1 x	Blur can 1 FS eff 1	Blur can 1 FS eff 1		Blur can 1 FS eff 1	Blur can 1 FS eff 1	
	EnFnLn 1	EnFnLn 1		EnFnLn 1	EnFnLn 1	EnFnLn 1		EnFnLn 1	EnFnLn 1	
48 x 48	22 x 22	22 x 22		22 x 22	22 x 22	22 x 22		22 x 22	22 x 22	
10 / 2.5	<b>4 / .5</b>	<b>4 / .5</b>		<b>4 / 1.5</b>	<b>4 / 1.5</b>	<b>4 / 1.5</b>		<b>4 / .5</b>	<b>4 / 1.5</b>	
15 sls	20 sls/2aq	20 sls/2aq		20 sls/2aq	20 sls/3aq	22 sl / 1 acq		20 sls/3aq	20 sls/3aq	
256 x 128	256 x 224	256 x 192		256 x 224	256 x 224	256 x 224		256 x 224	256 x 224	
1 nex	2 nex	2 nex		2 nex	2 nex	2 nex		2 nex	2 nex	
pfov 1	pfov 1	pfov 1		pfov 1	pfov 1	pfov 1		pfov 1	pfov 1	
frq def	frq unswap	frq unswap		frq unswap	frq unswap	frq unswap		frq unswap	frq unswap	
0:32	2:12 / 2aqs	5:24 / 2aqs		2:12 / 2aqs	3:30 / 3aqs	2:52		3:30 / 3aqs	3:30 / 3aqs	