

Wrist Osteo/Tumor/Abscess (Adult) SIGNA Architect

Technical Instructions

3-PLANE LOC

AX T2 cl fat

COR T2 cl fat

SAG T2 cl fat

AX T1

COR T1

--Tumor ONLY--

Pre AX T1 FAT

--Post Contrast--

+C COR T1 FAT

+C AX T1 FAT

--optional--

Sag T2 FAT PROPELLER

Cor T2 FAT PROPELLER

Ax T2 FAT PROPELLER

Ax PD FS PROPELLER

COR PD IDEAL

3-PLANE LOC

Details tab

Protocol notes

3-PLANE LOC

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	3-PLANE

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	Seq, EDR, Fast, SS, ARC
Pulse Sequence:	Spin Echo
ARC Phase:	2.00

Image Recon

Recon DL Strength:	Off
Intensity Correction:	NONE
Intensity Filter:	None
Save Original:	Off

SCAN RANGE

FOV:	30.0
Slice Thickness:	5.00
Slice Spacing:	5.0
Overlap Locations:	0
No Phase Wrap:	1.00

ACQ TIMING

Acq Voxel Size:	0.8 X 1.2 X 5.0
Freq:	384
Phase:	256
Freq DIR:	Unswap
# of Acq. Before Pause:	0
Phase FOV:	1.00
Auto Shim:	Auto
Phase Correct:	Off
RF Drive Mode:	Preset
Excitation Mode:	Selective

SCAN TIMING

TE:	85.0
Number of Echoes:	1
TR:	Minimum
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

3D Geometry Correction:	Off
Anatomical Region:	Wrist joint

USER CVS

User CV1:	1.00
User CV2:	240.00

User CV13:

1.00

Non-Viewable Parameters

$\tilde{n}$

AX T2 cl fat

Details tab

Protocol notes

AX T2 cl fat

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	OBLIQUE

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	EDR, TRF, Fast, FR
Pulse Sequence:	FSE-XL

Image Recon

Recon DL Strength:	Medium
Intensity Correction:	SCENIC
Intensity Filter:	None
Save Original:	Off

SCAN RANGE

FOV:	10.0
Slice Thickness:	3.00
Slice Spacing:	0.5
Overlap Locations:	0
Number of Slices:	15
No Phase Wrap:	1.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.4 X 3.0
Freq:	288
Phase:	256
Freq DIR:	Unswap
NEX:	1.00
# of Acq. Before Pause:	0
Phase FOV:	0.90
Auto Shim:	Auto
Phase Correct:	On
RF Drive Mode:	Quadrature (CP)
Excitation Mode:	Selective
Fat Shift DIR:	Normal

SCAN TIMING

Flip Angle:	111.0
TE:	80.0
Number of Echoes:	1
TR:	3500.0
Echo Train Length:	19
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

SAT

Fat/Water Saturation:	Fat Classic
-----------------------	-------------

USER CVS

User CV6:	None
User CV9:	None
User CV15:	None
User CV18:	None
User CV19:	1.00
User CV21:	None
User CV26:	1.00
User CV41:	None

Non-Viewable Parameters

ñ

COR T2 cl fat

Details tab

Protocol notes

COR T2 cl fat

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	OBLIQUE

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	EDR, TRF, Fast, FR
Pulse Sequence:	FSE-XL

Image Recon

Recon DL Strength:	Medium
Intensity Correction:	SCENIC
Intensity Filter:	None
Save Original:	Off

SCAN RANGE

FOV:	10.0
Slice Thickness:	2.00
Slice Spacing:	0.2
Overlap Locations:	0
Number of Slices:	15
No Phase Wrap:	1.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.4 X 2.0
Freq:	288
Phase:	256
Freq DIR:	Unswap
NEX:	1.00
# of Acq. Before Pause:	0
Phase FOV:	0.90
Auto Shim:	Auto
Phase Correct:	On
RF Drive Mode:	Quadrature (CP)
Excitation Mode:	Selective
Fat Shift DIR:	Normal

SCAN TIMING

Flip Angle:	111.0
TE:	80.0
Number of Echoes:	1
TR:	3500.0
Echo Train Length:	19
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

SAT

Fat/Water Saturation:	Fat Classic
-----------------------	-------------

USER CVS

User CV6:	None
User CV9:	None
User CV15:	None
User CV18:	None
User CV19:	1.00
User CV21:	None
User CV26:	1.00
User CV41:	None

Non-Viewable Parameters

ñ

SAG T2 cl fat

Details tab

Protocol notes

SAG T2 cl fat

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	OBLIQUE

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	EDR, TRF, Fast, FR
Pulse Sequence:	FSE-XL

Image Recon

Recon DL Strength:	Medium
Intensity Correction:	SCENIC
Intensity Filter:	None
Save Original:	Off

SCAN RANGE

FOV:	12.0
Slice Thickness:	3.00
Slice Spacing:	0.5
Overlap Locations:	0
Number of Slices:	15
No Phase Wrap:	1.00

ACQ TIMING

Acq Voxel Size:	0.4 X 0.5 X 3.0
Freq:	288
Phase:	256
Freq DIR:	Unswap
NEX:	1.00
# of Acq. Before Pause:	0
Phase FOV:	0.90
Auto Shim:	Auto
Phase Correct:	On
RF Drive Mode:	Quadrature (CP)
Excitation Mode:	Selective
Fat Shift DIR:	Normal

SCAN TIMING

Flip Angle:	111.0
TE:	80.0
Number of Echoes:	1
TR:	3500.0
Echo Train Length:	19
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

SAT

Fat/Water Saturation: Fat Classic

USER CVS

User CV6:	None
User CV7:	None
User CV9:	None
User CV15:	None
User CV18:	None
User CV19:	1.00
User CV21:	None
User CV26:	1.00
User CV41:	None

Non-Viewable Parameters

ñ

AX T1

Details tab

Protocol notes

AX T1

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	AXIAL

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	EDR, TRF, Fast, ARC
Pulse Sequence:	FSE-XL
ARC Phase:	1.50

Image Recon

Recon DL Strength:	Medium
Intensity Correction:	SCENIC
Intensity Filter:	None
Save Original:	Off

SCAN RANGE

FOV:	10.0
Slice Thickness:	3.00
Slice Spacing:	0.5
Overlap Locations:	0
No Phase Wrap:	1.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.4 X 3.0
Freq:	288
Phase:	256
Freq DIR:	A/P
NEX:	1.00
Phase FOV:	1.00
Auto Shim:	Auto
Phase Correct:	On
RF Drive Mode:	Quadrature (CP)
Excitation Mode:	Selective
Fat Shift DIR:	Normal (A)

SCAN TIMING

Flip Angle:	111.0
TE:	16.0
Number of Echoes:	1
TR:	800.0
Echo Train Length:	4
Receiver Bandwidth:	50.00

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

USER CVS

User CV6:	None
User CV9:	None
User CV15:	None
User CV16:	None
User CV18:	None
User CV21:	None
User CV26:	1.00
User CV41:	None

Non-Viewable Parameters

ñ

COR T1

Details tab

Protocol notes

COR T1

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	CORONAL

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	EDR, TRF, Fast, ARC
Pulse Sequence:	FSE-XL
ARC Phase:	1.50

Image Recon

Recon DL Strength:	Medium
Intensity Correction:	SCENIC
Intensity Filter:	None
Save Original:	Off

SCAN RANGE

FOV:	10.0
Slice Thickness:	2.00
Slice Spacing:	0.2
Overlap Locations:	0
No Phase Wrap:	1.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.4 X 2.0
Freq:	288
Phase:	256
Freq DIR:	S/I
NEX:	1.00
Phase FOV:	1.00
Auto Shim:	Auto
Phase Correct:	On
RF Drive Mode:	Quadrature (CP)
Excitation Mode:	Selective
Fat Shift DIR:	Normal (I)

SCAN TIMING

Flip Angle:	111.0
TE:	16.0
Number of Echoes:	1
TR:	500.0
Echo Train Length:	4
Receiver Bandwidth:	50.00

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

USER CVS

User CV6:	None
User CV9:	None
User CV15:	None
User CV16:	None
User CV18:	None
User CV21:	None
User CV26:	1.00
User CV41:	None

Non-Viewable Parameters

ñ

--Tumor ONLY--

Details tab

Protocol notes

--Tumor ONLY--

PATIENT POSITION

Coil Configuration:

Neo16chFlexS

Plane:

3-PLANE

IMAGING PARAMETERS

Imaging Mode:

2D

Imaging Options:

Seq, EDR, Fast, SS

Pulse Sequence:

Spin Echo

Image Recon

Intensity Correction:

NONE

Intensity Filter:

None

SCAN RANGE

FOV:

30.0

Slice Thickness:

5.0

Slice Spacing:

5.0

No Phase Wrap:

1.00

ACQ TIMING

Acq Voxel Size:

0.8 X 1.2 X 5.0

Freq:

384

Phase:	256
Freq DIR:	Unswap
# of Acq. Before Pause:	0
Phase FOV:	1.00
Auto Shim:	Auto
Phase Correct:	Off
RF Drive Mode:	Preset
Excitation Mode:	Selective

SCAN TIMING

TE:	85.0
TR:	Minimum
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

3D Geometry Correction:	Off
Anatomical Region:	Wrist joint

GATING/TRIGGER

Auto Trigger Type:	Off
--------------------	-----

USER CVS

User CV1:	1.0
User CV2:	240.00

User CV13:	1.00
User CV15:	None
User CV39:	1.00

Non-Viewable Parameters

ñ

Pre AX T1 FAT

Details tab

Protocol notes

Pre AX T1 FAT

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	AXIAL

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	EDR, TRF, Fast, ARC
Pulse Sequence:	FSE-XL
ARC Phase:	1.50

Image Recon

Recon DL Strength:	Medium
Intensity Correction:	SCENIC
Intensity Filter:	None
Save Original:	Off

SCAN RANGE

FOV:	10.0
Slice Thickness:	3.00
Slice Spacing:	0.5
Overlap Locations:	0
No Phase Wrap:	1.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.4 X 3.0
Freq:	288
Phase:	256
Freq DIR:	A/P
NEX:	1.00
Phase FOV:	1.00
Auto Shim:	Auto
Phase Correct:	On
RF Drive Mode:	Quadrature (CP)
Excitation Mode:	Selective
Fat Shift DIR:	Normal (A)

SCAN TIMING

Flip Angle:	111.0
TE:	16.0
Number of Echoes:	1
TR:	800.0
Echo Train Length:	4
Receiver Bandwidth:	50.00

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

Fat/Water Saturation:***Fat*****USER CVS**

User CV6:	None
User CV9:	None
User CV15:	None
User CV16:	None
User CV18:	None
User CV19:	1.00
User CV21:	None
User CV26:	1.00
User CV41:	None

Non-Viewable Parameters

ñ

--Post Contrast--

Details tab

Protocol notes

--Post Contrast--

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	3-PLANE

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	Seq, EDR, Fast, SS
Pulse Sequence:	Spin Echo

Image Recon

Intensity Correction:	NONE
Intensity Filter:	None

SCAN RANGE

FOV:	30.0
Slice Thickness:	5.0
Slice Spacing:	5.0
No Phase Wrap:	1.00

ACQ TIMING

Acq Voxel Size:	0.8 X 1.2 X 5.0
Freq:	384

Phase:	256
Freq DIR:	Unswap
# of Acq. Before Pause:	0
Phase FOV:	1.00
Auto Shim:	Auto
Phase Correct:	Off
RF Drive Mode:	Preset

SCAN TIMING

TE:	85.0
TR:	Minimum
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

3D Geometry Correction:	Off
Anatomical Region:	Wrist joint

GATING/TRIGGER

Auto Trigger Type:	Off
--------------------	-----

USER CVS

User CV1:	1.0
User CV2:	240.00
User CV13:	1.00

TR Min: -
TR Max: -

Non-Viewable Parameters

ñ

+C COR T1 FAT

Details tab

Protocol notes

+C COR T1 FAT

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	CORONAL

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	EDR, TRF, Fast, ARC
Pulse Sequence:	FSE-XL
ARC Phase:	1.50

Image Recon

Recon DL Strength:	Medium
Intensity Correction:	SCENIC
Intensity Filter:	None
Save Original:	Off

SCAN RANGE

FOV:	10.0
Slice Thickness:	2.00
Slice Spacing:	0.2
Overlap Locations:	0
No Phase Wrap:	1.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.4 X 2.0
Freq:	288
Phase:	256
Freq DIR:	S/I
NEX:	1.00
Phase FOV:	1.00
Auto Shim:	Auto
Phase Correct:	On
RF Drive Mode:	Quadrature (CP)
Excitation Mode:	Selective
Fat Shift DIR:	Normal (I)

SCAN TIMING

Flip Angle:	111.0
TE:	16.0
Number of Echoes:	1
TR:	800.0
Echo Train Length:	4
Receiver Bandwidth:	50.00

CONTRAST

Contrast Yes/No:	Yes
------------------	-----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

Fat/Water Saturation:***Fat*****USER CVS**

User CV6:	None
User CV9:	None
User CV15:	None
User CV16:	None
User CV18:	None
User CV19:	1.00
User CV21:	None
User CV26:	1.00
User CV41:	None

Non-Viewable Parameters

ñ

+C AX T1 FAT

Details tab

Protocol notes

+C AX T1 FAT

PATIENT POSITION

Coil Configuration:Neo16chFlexS

Plane:AXIAL

IMAGING PARAMETERS

Imaging Mode:2D

Imaging Options:EDR, TRF, Fast, ARC

Pulse Sequence:FSE-XL

ARC Phase:1.50

Image Recon

Recon DL Strength:Medium

Intensity Correction:SCENIC

Intensity Filter:None

Save Original:Off

SCAN RANGE

FOV:10.0

Slice Thickness:3.00

Slice Spacing:0.5

Overlap Locations:0

No Phase Wrap:1.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.4 X 3.0
Freq:	288
Phase:	256
Freq DIR:	A/P
NEX:	1.00
Phase FOV:	1.00
Auto Shim:	Auto
Phase Correct:	On
RF Drive Mode:	Quadrature (CP)
Excitation Mode:	Selective
Fat Shift DIR:	Normal (A)

SCAN TIMING

Flip Angle:	111.0
TE:	16.0
Number of Echoes:	1
TR:	800.0
Echo Train Length:	4
Receiver Bandwidth:	50.00

CONTRAST

Contrast Yes/No:	Yes
------------------	-----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

Fat/Water Saturation:***Fat*****USER CVS**

User CV6:	None
User CV9:	None
User CV15:	None
User CV16:	None
User CV18:	None
User CV19:	1.00
User CV21:	None
User CV26:	1.00
User CV41:	None

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
Non-Viewable Parameters

ñ

--optional--

Details tab


Protocol notes



--optional--

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	3-PLANE

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

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	Seq, EDR, Fast, SS
Pulse Sequence:	Spin Echo


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


Image Recon

Intensity Correction:	NONE
Intensity Filter:	None

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

SCAN RANGE

FOV:	30.0
Slice Thickness:	5.0
Slice Spacing:	5.0
No Phase Wrap:	1.00



ACQ TIMING

Acq Voxel Size:	0.8 X 1.2 X 5.0
Freq:	384

Phase:	256
Freq DIR:	Unswap
# of Acq. Before Pause:	0
Phase FOV:	1.00
Auto Shim:	Auto
Phase Correct:	Off
RF Drive Mode:	Preset

SCAN TIMING

TE:	85.0
TR:	Minimum
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

3D Geometry Correction:	Off
Anatomical Region:	Wrist joint

GATING/TRIGGER

Auto Trigger Type:	Off
--------------------	-----

USER CVS

User CV1:	1.0
User CV2:	240.00
User CV13:	1.00

TR Min: -
TR Max: -

Non-Viewable Parameters	$\tilde{n}$
--------------------------------	-------------

Sag T2 FAT PROPELLER

Details tab

Protocol notes

Sag T2 FAT PROPELLER

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	OBLIQUE

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	EDR, TRF, Fast, FR, ARC
Pulse Sequence:	PROPELLER
ARC Phase:	2.00

Image Recon

Recon DL Strength:	Medium
Intensity Correction:	PURE
Intensity Filter:	None
Save Original:	Off

SCAN RANGE

FOV:	10.0
Slice Thickness:	3.00
Slice Spacing:	0.5
Overlap Locations:	0
Number of Slices:	12

No Phase Wrap:

2.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.3 X 3.0
Freq:	320
Freq DIR:	A/P
NEX:	2.00
# of Acq. Before Pause:	0
Auto Shim:	On
Phase Correct:	Off
RF Drive Mode:	Preset
Excitation Mode:	Selective
Fat Shift DIR:	Normal (A)

SCAN TIMING

Flip Angle:	111.0
TE:	80.0
Number of Echoes:	1
TR:	1781.0
Echo Train Length:	19
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

SAT

Fat/Water Saturation:	Fat
-----------------------	-----

GATING/TRIGGER

Resp. Trigger Point:	30
Resp. Trigger Window:	10

USER CVS

User CV6:	None
User CV15:	None
User CV19:	0.80
User CV26:	1.00
User CV40:	None
TR Min:	400.0
TR Max:	11000.0

Non-Viewable Parameters

ñ

Cor T2 FAT PROPELLER

Details tab

Protocol notes

Cor T2 FAT PROPELLER

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	OBLIQUE

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	EDR, TRF, Fast, FR, ARC
Pulse Sequence:	PROPELLER
ARC Phase:	2.00

Image Recon

Recon DL Strength:	Medium
Intensity Correction:	SCENIC
Intensity Filter:	None
Save Original:	Off

SCAN RANGE

FOV:	10.0
Slice Thickness:	3.00
Slice Spacing:	0.5
Overlap Locations:	0
Number of Slices:	12

No Phase Wrap:

2.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.3 X 3.0
Freq:	320
Freq DIR:	A/P
NEX:	2.00
# of Acq. Before Pause:	0
Auto Shim:	On
Phase Correct:	Off
RF Drive Mode:	Preset
Excitation Mode:	Selective
Fat Shift DIR:	Normal (A)

SCAN TIMING

Flip Angle:	111.0
TE:	80.0
Number of Echoes:	1
TR:	1781.0
Echo Train Length:	19
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

SAT

Fat/Water Saturation:	Fat
-----------------------	-----

GATING/TRIGGER

Resp. Trigger Point:	30
Resp. Trigger Window:	10

USER CVS

User CV6:	None
User CV15:	None
User CV19:	0.80
User CV26:	1.00
User CV41:	None
TR Min:	400.0
TR Max:	11000.0

Non-Viewable Parameters

ñ

Ax T2 FAT PROPELLER

Details tab

Protocol notes

Ax T2 FAT PROPELLER

PATIENT POSITION

Coil Configuration:Neo16chFlexS

Plane:OBLIQUE

IMAGING PARAMETERS

Imaging Mode:2D

Imaging Options:EDR, TRF, Fast, FR, ARC

Pulse Sequence:PROPELLER

ARC Phase:2.00

Image Recon

Recon DL Strength:Medium

Intensity Correction:SCENIC

Intensity Filter:None

Save Original:Off

SCAN RANGE

FOV:10.0

Slice Thickness:3.00

Slice Spacing:0.5

Overlap Locations:0

Number of Slices:12

No Phase Wrap:

2.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.3 X 3.0
Freq:	320
Freq DIR:	A/P
NEX:	2.00
# of Acq. Before Pause:	0
Auto Shim:	On
Phase Correct:	Off
RF Drive Mode:	Preset
Excitation Mode:	Selective
Fat Shift DIR:	Normal (A)

SCAN TIMING

Flip Angle:	111.0
TE:	80.0
Number of Echoes:	1
TR:	1781.0
Echo Train Length:	19
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

SAT

Fat/Water Saturation:	Fat
-----------------------	-----

USER CVS

User CV6:	None
User CV15:	None
User CV19:	0.80
User CV26:	1.00
User CV41:	None
TR Min:	400.0
TR Max:	11000.0

Non-Viewable Parameters

ñ

Ax PD FS PROPELLER

Details tab

Protocol notes

Ax PD FS PROPELLER

PATIENT POSITION

Coil Configuration:Neo16chFlexS

Plane:OBLIQUE

IMAGING PARAMETERS

Imaging Mode:2D

Imaging Options:EDR, TRF, Fast, FR, ARC

Pulse Sequence:PROPELLER

ARC Phase:2.00

Image Recon

Recon DL Strength:Medium

Intensity Correction:PURE

Intensity Filter:None

Save Original:Off

SCAN RANGE

FOV:10.0

Slice Thickness:3.00

Slice Spacing:0.5

Overlap Locations:0

Number of Slices:12

No Phase Wrap:

2.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.3 X 3.0
Freq:	288
Freq DIR:	R/L
NEX:	2.00
# of Acq. Before Pause:	0
Auto Shim:	On
Phase Correct:	Off
RF Drive Mode:	Preset
Excitation Mode:	Selective
Fat Shift DIR:	Normal (L)

SCAN TIMING

Flip Angle:	80.0
TE:	63.7
Number of Echoes:	1
TR:	1402.0
Echo Train Length:	16
Receiver Bandwidth:	62.50

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

SAT

Fat/Water Saturation:	Fat Classic
-----------------------	-------------

GATING/TRIGGER

Resp. Trigger Point:	30
Resp. Trigger Window:	10

USER CVS

User CV6:	None
User CV15:	None
User CV19:	1.00
User CV26:	1.00
User CV40:	None
TR Min:	400.0
TR Max:	11000.0

Non-Viewable Parameters



COR PD IDEAL

Details tab

Protocol notes

COR PD IDEAL

PATIENT POSITION

Coil Configuration:	Neo16chFlexS
Plane:	OBLIQUE

IMAGING PARAMETERS

Imaging Mode:	2D
Imaging Options:	TRF, Fast, FR, IDEAL
Pulse Sequence:	FSE-XL
IDEAL:	3

Image Recon

Intensity Correction:	NONE
Intensity Filter:	None

SCAN RANGE

FOV:	10.0
Slice Thickness:	4.0
Slice Spacing:	1.0
No Phase Wrap:	2.00

ACQ TIMING

Acq Voxel Size:	0.3 X 0.5 X 4.0
-----------------	-----------------

<i>Freq:</i>	320
Phase:	192
Freq DIR:	Unswap
NEX:	0.50
Auto Shim:	Auto
Phase Correct:	On
RF Drive Mode:	Quadrature

SCAN TIMING

TE:	30.0
TR:	2000.0
Echo Train Length:	6
Receiver Bandwidth:	41.67

CONTRAST

Contrast Yes/No:	No
------------------	----

OTHERS

Anatomical Region:	Wrist joint
--------------------	-------------

GATING/TRIGGER

Auto Trigger Type:	Off
--------------------	-----

USER CVS

User CV6:	None
User CV8:	None
User CV9:	None

User CV17:	1.00
User CV21:	None
User CV22:	1.00
TR Min:	-
TR Max:	-

Non-Viewable Parameters

ñ